

PRODUCT NAME:


BREATHABLE ANKLE SUPPORT WITH THERMOPLASTIC PLATES – ORTHESIS FOR ANKLE CONTROL

REFERENCE:

EST-084

SIZES:

SIZES	PERIMETER CM
1	<19
2	19-20
3	20-23
4	23-25



MANUFACTURER:

ORLIMAN, S.L.U. CIF: B-96122510
 C/ AUSIAS MARCH Nº 3. POLÍGONO INDUSTRIAL LA POBLA – L'ELIANA.
 46185 LA POBLA DE VALLBONA – VALENCIA – SPAIN
 SPANISH MEDICAL DEVICE OPERATION LICENCE NUMBER: 20-PS

PRODUCT CLASSIFICATION (Council Directive 93/42/EEC (as amended by Council Directive 2007/47/EC))

CLASS: I – (Rule 1)

Manufacturing type: Mass-Produced

PRODUCT CHARACTERISTICS:

Ankle stabiliser made in breathable semi-rigid fabric and designed to be able to opt for different indications, depending on the chosen configuration, as we can select from three types of components: conventional ankle support, ankle support with pronosupine webbing and ankle support with webbing and malleolar plates (internally and externally identified).

Colours: Black.

PRODUCT PICTURE:



INDICATIONS:

- Conservative treatment in breakage of ankle ligaments.
- As postoperative protection in reconstructive ligament interventions.
- Injury of medial and lateral ankle ligaments.
- Chronic ankle instability.
- Post-operative rehabilitation. Tendinopathies.

MATERIALS:

- Components: LATEX FREE
- Structure materials: Inside: 80% Cotton, 20% Polyester / Intermediate: 100% Polyester / Outside: 100% Polyester
- Extra-structure materials: PLATES: Polyethylene 100%.
- Fastening or catch elements, if it has any: Hook closure.





TECHNICAL DATASHEET

IM-30-04/1

Edition: 22/03/18

Page: 2 of 2

RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Stick the velcros to each other (if the orthosis has any), wash periodically by hand with a damp cloth and a neutral soap. To dry the orthosis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.



ADDITIONAL INFORMATION:

Guarantee: 6 month.



Difarza, s.l. CIF B96386875 Tel.: 963363215 Fax: 963671551
email: disalud@ono.com
www.disalud.com

C/Río Ulla, 5 (entrada por Juan Verdeguer, 25) 46024 Valencia.
Empresa colaboradora de servicios médicos y de prevención de riesgos.
Distribución de productos sanitarios, clínicos y ortopédicos desde 1980.
Prevención, Seguridad y Salud Laboral.



1112&10#%&2.*3



1112#(3#.!#&10#%&2.*3



!''#%&' (%) *+, -#\$, .!#



/ &!0#%&