TECHNICAL DATASHEET

IM-30-04/1

Edition: 13/05/13

Page: 1 of 2

PRODUCT NAME:

NEOPRENE THUMB WRIST SUPPORT- THERMO-MED®
(Thermo-compression Orthosis)

REFERENCE:

4604 (Blue) – 5604 (Beige)

SIZES:

SIZES	PERIMETER CM
XS/1	15-17
S/2	17-19
M/3	19-21
L/4	21-23

MANUFACTURER:

ORLIMAN, S.L.U. CIF: B-96122510
C/ AUSIAS MARCH Nº 3. POLÍGONO INDUSTRIAL LA POBLA – L'ELIANA.
46185 LA POBLA DE VALLBONA – VALENCIA – SPAIN
SPANISH MEDICAL DEVICE OPERATION LICENCE NUMBER: 20-PS

PRODUCT CLASSIFICATION (Council Directive 93/42/EEC (as amended by Council Directive 2007/47/EC))

CLASS: I – (Rule 1)

Manufacturing type: Mass-Produced

PRODUCT CHARACTERISTICS:

3 mm neoprene wrist support, with Velcro fastening and moldable thumb splint.

Colours:

- Blue: 4604
- Beige: 5604

INDICATIONS:

-Immobilisation and protection of the carpometacarpal and etacarpophalangeal joints.
-Slight wrist instabilities.
-Protection during sporting activities.

MATERIALS:

- Components: Contain LATEX, natural rubber.
- Fabric: Inside: Polyamide 100%; Intermediate: Polychloroprene 100%; Outside: Polyamide 100%.
- Fastening or catch elements, if it has any: Hook closure.
- Splint: Aluminium.

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Edition: 13/05/13

Page: 2 of 2

RECOMMENDATIONS FOR PRESERVATION AND CLEANING

Stick the velcros to each other (if the orthosis has any), wash periodically by hand with a damp cloth and a neutral soap. To dry the orthosis, use a dry towel to absorb most of the dampness and leave to dry at room temperature. Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

Instructions for the extraction and fitting of lateral reinforcements inserted in knee supports – for ease in washing of products. (If there are any)

Pull at both ends of knee support until the hinge appears at the upper openings. Once product has been washed and completely dry, reinsert the reinforcements.

Precautions: In order to ensure the therapeutic effectiveness during normal use of knee supports, the reinforcements must never be handled or extracted from their correct position, except for washing or under specific orders from the prescribing doctor or orthopaedic technician.

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